


# Operoll - inflatable cylinder, must-have in every operating room

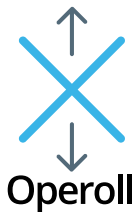

The Operoll is an inflatable cylinder, tested for **limited reuse** (3-6 months). Thanks to its simple **cylinder shape** and easy-to-use deflation/inflation valve, the Operoll provides optimal positioning for a wide range of applications, such as prone positioned epidural in pain clinics, lateral positioning (AAA and kidney procedures), difficult intubations, etc.

Note: The Operoll is an **all-in-one patient support**, but when you need more stability, 100% hygiene or a solution for heavy weight patients, we recommend the IXS.



## ENSURE PATIENT SAFETY & SAFE THE PLANET

No product lasts forever, especially if it is frequently cleaned with disinfection products. That's why it is important to replace your Operoll before it is defect.



### How long can I safely use the Operoll?

The Operoll is tested for 1000 inflations and has a standard warranty of 3 months. After registration (via the Ergotrics website), the warranty is extended to 6 months from invoice date.

### We can recycle your old product!


When you return your used Operoll within 6 months after purchase, **you get 55€ discount** on your new purchase!

Visit our website  
for more info:  
[www.ergotrics.com/return](http://www.ergotrics.com/return)




Thanks to the simple **cylinder shape** and easy-to-use deflation/inflation valve, the Operoll can be used for **MULTIPLE APPLICATIONS, FOR EXAMPLE:**


- **During intubation, it is sometimes difficult to insert the tube into the trachea**, for example when the patient has a high BMI, a short neck, small mandible, small mouth opening or a stiff neck. With the Operoll, the optimal intubation position can be set for each patient: you can simply inflate or gently deflate the cushion to the desired height.

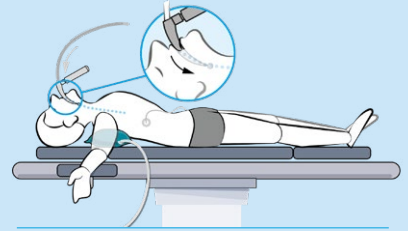

- **The most common application of the Operoll is in the lateral position.** You place the Operoll under the armpit or the waist and inflate it during the procedure to reach the surgical goal (e.g. heart of kidney). This application is commonly known as the Kidneyroll (or Nierenrolle).

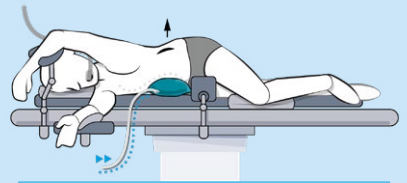

- **Operoll for prone positioned epidural infiltration, nerve blocks or implantation of epidural electrodes (Spinal Cord Stimulation).** Painless accessibility of the epidural space is a challenge. For optimal treatment, the lumbar spine needs to delordosate. Operoll helps to bring the patient into the right position for ideal treatment. When the Operoll is inflated, the patient's hollow back is straightened creating a wider interlaminar window.

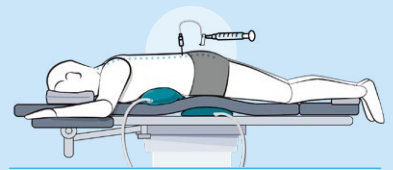

## SMART DESIGN with multiple **ADVANTAGES** for users:



**Adjustable** (inflate/deflate)



**Safety valve** (avoids overpressure with umbrella valves and warning whistle tone)



**Double cover to avoid leaks** (airtight high frequency welding inside)



**Biocompatible cover** (Oekotex certified) that can be used directly on skin



**X-ray permeable**



**Better ergonomics** for healthcare workers by using compressed air

**Product info or demo?**

info@ergotrics.com • www.ergotrics.com